

Mepilex Border Post-OP Ag Application Quiz Key

1. When applying Mepilex Border Post-Op / Ag the skin should be completely dry.
True
False
2. Choose a dressing where the wound pad overlaps the incision by at least ____ cm on each side
 - a. 3 cm
 - b. 1-2 cm
 - c. 0.5 cm
3. The dressing should never be stretched during application.
True
False
4. The dressing should be changed every 3rd day.
True
False
5. Choose all answers that are correct.
Clinical indications to change the dressing include:
 - a. The dressing pad is saturated
 - b. There is strike through and the dressing doesn't look clean and dry
 - c. Any signs of infection
 - d. All of the above
6. Put the steps of the dressing application in the correct order.
 - a. Ensure the skin is clean and completely dry
 - b. Choose a dressing that overlaps the incision by at least 1-2 cm on each side
 - c. Repeat for the smaller film and reposition if necessary
 - d. Remove the larger of the remaining films continuously while applying the dressing
 - e. Remove the middle part of the release film and apply on the right (up) position
 - f. Finalize the application by smoothing the dressing borders for optimal adhesion

Answer

- a. Ensure the skin is clean and completely dry
- b. Choose a dressing that overlaps the incision by at least 1-2 cm on each side
- c. Remove the middle release film and apply to in the right up position
- d. Remove the larger of the remaining films continuously while applying the dressing
- e. Repeat for the smaller film and reposition if necessary
- f. Finalize the application by smoothing the dressing borders for optimal adhesion